

CATTLE IMMOBILIZER

INSTRUCTIONS

please read this entire manual carefully before using the device



THE SAFE WAY TO PERFORM

- Castration
- Dehorning
- Branding
- Foot trimming
- Nose ringing
- Clipping
- Earmarking
- Difficult calving
- Mastitis Treatment
- Vaccination

Function

Cattle from cows, calves to bulls are immobilized through electric rectal probe stunning. Safe electrical currents are induced into the soft tissue of the animal causing a temporary and completely reversible paralysis of the muscle. Alternating between muscle contraction and relaxation at high frequencies allow for effective control of the beast while not inducing excessive muscle fatigue or damage. In terms of safety the Scintex cattle immobilizer provides benefits over large animal anesthesia or the alternative of having only crush or cradle restraint.

Adjustability

Adjust the cattle immobilizer using the twist knob on the outside to easily dial in the required setting for each animal being restrained.

Insertion is performed with the immobilizer switched off, then by increasing the current an optimum setting for that beast can be quickly found. If during the procedure the beast is too active simply stop and increase the setting.

Batteries

Operates on household rechargeable AA or normal AA batteries. If the unit goes flat new batteries to keep you working are readily available, 10 hours operation on a single set of AA batteries.

Maintain

Water resistant – easy to clean after use, just wipe down with a damp cloth

